**PROSOMNUS®**  
SLEEP TECHNOLOGIES

## ProSomnus® Sleep and Snore Devices

### INSTRUCTIONS FOR USE

**ProSomnus EVO™**

**ProSomnus [IA]**

**ProSomnus [CA]**

**ProSomnus [PH]**

**ProSomnus MicroO<sub>2</sub>®**

U.S. Patent Nos. 9,808,327|9,820,882|9,949,868|10,213,280|10,603,207

<https://IFU.ProSomnus.com>  
**PRO3-267-A (March 2021)**

**ProSomnus Sleep Technologies**  
**5860 West Las Positas Blvd., Suite 25**  
**Pleasanton, CA 94588, USA**  
**Phone +1 844 537 5337**  
**ProSomnus.com**

**Market Access (AUS) Pty Ltd**  
**Australian Sponsor**  
**810 Pacific Highway**  
**Gordon, NSW 2072, Australia**

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#### ENGLISH

##### Product Information

This is your custom prescribed ProSomnus Sleep and Snore Device ordered by your Dentist or Physician. Your Dentist or Physician will review the proper techniques and instructions for adjusting, maintenance and cleaning your custom sleep device. A supplemental User Manual is available at <https://IFU.ProSomnus.com>.

##### Indications

The ProSomnus Sleep and Snore Devices are intended to reduce night-time snoring and mild to moderate Obstructive Sleep Apnea (OSA) in adults.

##### Contraindications

The device is contraindicated for patients who:

- Have Central Sleep Apnea
- Have severe respiratory disorders
- Have loose teeth or advance periodontal disease
- Are under 18 years of age

There are known hazards with using an oral sleep apnea device. Have your Dentist or Physician review them with you.

##### Warnings

Use of the device may cause:

- Tooth movement or changes in dental occlusion
- Gingival or dental soreness
- Pain or soreness to the temporomandibular joints
- Obstruction of oral breathing
- Excessive salivation

##### Precautions

Patients should consider their medical history, including history of asthma, breathing, or respiratory disorders, or other relevant health problems, and contact the appropriate healthcare provider before using the device.

##### Storage

Keep out of reach of small children and pets.

The ProSomnus Sleep and Snore Device should be stored in a cool, dry place. Ensure the device is not exposed to extreme temperature in excess of 60°C/140°F.

The printed ProSomnus Sleep and Snore Device IFU may be requested at:  
+1 844 537 5337 or email [Info@ProSomnus.com](mailto:Info@ProSomnus.com)

**Caution:** Federal law restricts this custom device to sale by or on the order of a Dentist or Physician.

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